WOHLWEND AG

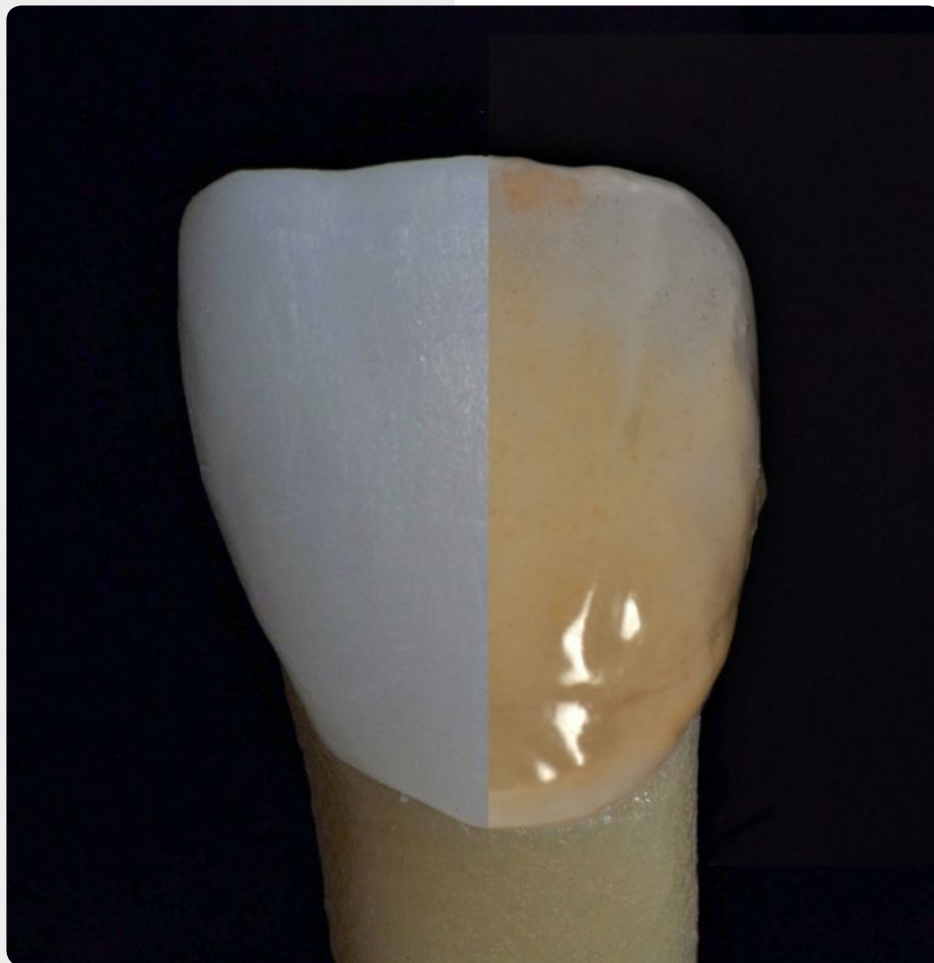
DENTAL TOTAL S.L.

ARTISTIC

Manual Técnico

Technical Manual

Vision
■ 3D ARTISTIC

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Introduction

The Vision-3D Artistic Paint-on Ceramic System consists of different components for refinement and finalisation that are coordinated with each other. This finishing system is ideal for zirconium and lithium disilicate materials. Also conventional metal-ceramics can be finalized with staining technique.

For small space requirements as well as monolithic crowns and bridges, the efficiently and quickly natural effects such as translucency, chroma, halo, mamelons, enamel cracks, opalescence and fluorescence can be achieved easily. The components of the Vision-3D Artistic System consist of fluorescent paints, sculpture masses and a special glaze mass.

The sculpture masses can be applied in a layer thickness of 0.1–0.4 mm over the individually painted surface. Due to their vital colour properties and their stability, three-dimensional color effects from the depths as well as refractive surface structures can be achieved with little effort.

- » Universal application
- » 3D layering effect
- » Increase of the chroma
- » Fluorescent
- » Colour brightening
- » Individual incisal effects
- » Reliable, reproducible results
- » Simple and economical handling

Classification

Vision-3D Artistic paint on ceramic is a Class IIa medical device.
The material is only for professional use in dental laboratories.
Processing should only be carried out by trained personnel.

Storage and processing instructions

- » Please store dry and at room temperature.
- » The pastes must not come into contact with water!
- » Mix pastes and masses well before each use!
- » Do not change Sculpture masses in consistency. If too much fluid is used the modelling stability is no longer guaranteed.
- » For a layer thicknesses over 0.4 mm on zircon or lithium disilicate, Vision-zircon or Vision-LDL layering ceramic should be used for the respective frame Material!

Important! Mix pastes and materials thoroughly before each use!

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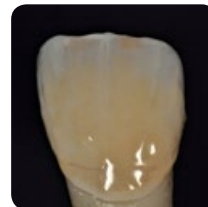
Componentes del Sistema Vision-3D Artistic

Vision-3D Artistic System Components

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Surface preparation

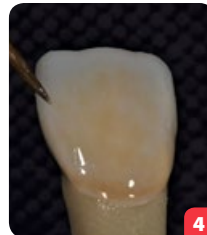
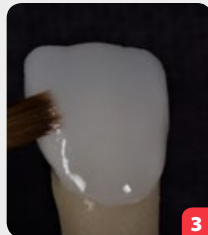
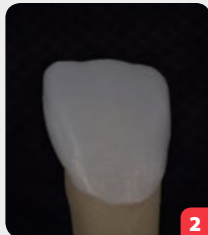
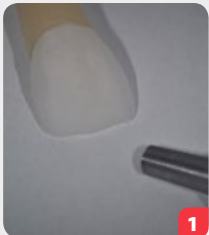
- » Glazed and smooth surfaces must be lightly roughened or beaded with glass beads. 50 μm and max.2 bar air pressure.
- » Then clean with steamer or distilled water in an ultrasonic cleaner.

Important! Mix stains well before use - store for longer can lead to a slight segregation between the paint powder and the fluid lead!

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Characterize / Individualize

- » Wet the surface with a little liquid and glaze.
- » Colouring in the desired colours and effects.
- » Colour fixation firing

Firing Parameter:

| Firing Brand Tipo de cocción | Start Temp T° Inicio | Drying Secado | Heat rate Rango Calentamiento | Final Temp. T° Final | Hold Time Mantenimiento | Vacio |
|--|-------------------------|------------------|----------------------------------|-------------------------|----------------------------|----------|
| Color fixation Zircon / Fijación de color Circón | 450°C | 2 Min | 40°C | 800°C | 1 Min | yes / si |
| Glaze firing Zirconia full anatomical Glasur / Cocción de glaseado de anatómica completa de Zirconia | 450°C | 2 Min | 40°C | 790°C | 1 Min | yes / si |
| Color fixation Lithium Disilicate / Fijación de color Disilicato de Litio | 450°C | 2 Min | 40°C | 755°C | 1 Min | yes / si |
| 3D Sculptur Mass Firing / Cocción de Masa de Escultura 3D | 400°C | 4 Min | 40°C | 750°C | 1 Min | yes / si |
| Glaze firing Lithium Disilicate /Sculptur masses / Cocción de glaseado de Disilicato de Litio /Masas de Escultura | 450°C | 2 Min | 40°C | 745°C | 1 Min | yes / si |

Glaze firing

Depending on the desired degree of gloss, a further glaze firing may be necessary.

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Sculpture Mass

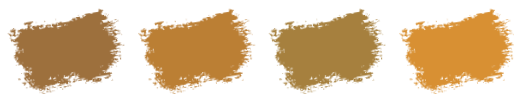
By overcoating the ceramic surface which has been individualized with Stains, a three-dimensional colour layer effect is achieved, which can hardly be distinguished to a regular multi-layering veneering. Especially for fully anatomical translucent zirconium as well as lithium disilicate when space is limited, an optically natural aesthetics is easily achieved. The optimum layer thickness of the Sculpture materials are 0.1 to max. 0.4 mm layer thickness. In addition regular layering materials (vision LDL or vision zircon) should be used according to the respective framework material.

The consistency of the sculpture materials should be similar to that of veneering composites.

- » Take out a small amount and knead it with a clean spatula.
- » Apply the desired mass to the surface of the ceramic restoration and modelate the mass with a clean brush into the desired shape.
- » With a dry fanned fine brush you can introduced crosswise and length-wise structures into the surface. (picture)
- » The sculptures masses are fired according to instructions.

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Fully anatomical zircon bridge individualized with Vision-3D Artistic

WOHLWEND AG

Wohlwend AG
Dental Manufaktur

Platta 52
FL-9488 Schellenberg
Phone: +423-373-4243
info@wohlwend-ag.com
www.wohlwend-ag.com

Santo Domingo, 56 Local
28917 La Fortuna / Leganés (MADRID)
Tlfno.: 91 610 16 01
info@dentaltotal.es
www.dentaltotal.es

CE 0483